



Product Name : Infant Weighing Scale

Product Model No : nice 1000 N

Trouble Shooting

**Doc no:
73-00-001**

1. General System Failure

No display on the control panel	Check A/C Supply Voltage
	Check if Power Cord is disconnected
	Check Circuit Breaker on the back side
	Otherwise, Contact nice Neötech
Bubble indicator not in the centre	Check the table where the machine is placed; Ensure that the table has no movements.
	Check the bubble level indicator; bubble should be in the center.
	Otherwise, Contact nice Neötech