



**Product Name : Fiber optic neonatal Transilluminator**

**Product Model No : nice 4050**

**Trouble Shooting**

**Doc no:  
73-00-018**

## 1. General System Failure

Sl. No	Problem	Cause	Remedy
1	AC IN indication LED is not turned on.	Power Failure	Check the unit is Plugged in to Main supply Check the mains are switched ON. Otherwise Contact nice Neötech.

## 2. System Fault Fibre Optic Neonatal Transilluminator

s.n o	Problem	Remedy
1.	Light not being emitted from the fibre optic panel	Check A/C Supply Voltage
		Check if Power Cord is disconnected
		Check panel is locked into illuminator correctly.
		Contact nice Neötech
2.	Illuminator LED Lamp is not turned ON	Check A/C Supply Voltage
		Check if Power Cord is disconnected
		Contact nice Neötech
3.	Illuminator LED Lamp is on but intensity is less	Check the percentage of intensity
		Replace Lamp
		Contact nice Neötech
4.	Fibre optic cable comes out	Check the locking mechanism
		Contact nice Neötech