



Product Name : Fibre Optic Blanket Infant Phototherapy

Product Model No : nice 4060

Trouble Shooting

**Doc no:
73-00-056**

1. General System Failure

s.no	Problem	Cause	Remedy
1.	AC IN indication LED is not turned on.	Power Failure	Check the unit is Plugged in to Main supply
			Check the mains are switched ON.
			Contact nice Neötech.

2. System Fault Fibre Optic Blanket Infant Phototherapy

s.no	Problem	Remedy
1.	Light not being emitted from the fibre optic panel	Check A/C Supply Voltage
		Check if Power Cord is disconnected
		Check panel is locked into illuminator correctly.
		Contact nice Neötech
2.	Illuminator LED Lamp is not turned ON	Check A/C Supply Voltage
		Check if Power Cord is disconnected
		Contact nice Neötech
3.	Illuminator LED Lamp is on but intensity is less	Check the percentage of intensity
		Replace Lamp
		Contact nice Neötech
4.	Fibre optic cable comes out	Check the locking mechanism
		Contact nice Neötech