



Product Name : Infant T-Piece Resuscitator

Product Model No : nice 5020

Trouble Shooting

**Doc no:
73-00-019**

1. General

S.no.	Problem	Remedy
1.	Unable to achieve the PIP and PEEP Pressure during the setup procedure	Check that the gas flow rate is set to 5-15L/min.
		Inspect the test lung for sign of damage.
		Ensure firm Connection between the gas supply, resuscitator, connectors and T-Piece circuit.
		Ensure the test lung is firmly connected to the T-Piece circuit.
		Confirm that the manometer shows zero with no gas flow.
		Check the maximum pressure relief is set correctly.
2.	The infant chest and upper abdomen are not rising during the inspiratory cycle	Confirm the good seal between the mask and infants face has been achieved.