

1. System Failure

Sl. No	Problem	Cause	Remedy
1	Monitor does not work.	No batteries fitted. Batteries fitted incorrectly. Batteries discharged.	Fit new batteries. Check that the batteries have been fitted as indicated in the battery compartment. Recharge / Fit new batteries.
2	No respiration signals.	Respiration Sensor not connected to the Monitor. Sensor pad has become loose on the skin.	Make sure that the Sensor connector is inserted in the Monitor's connector socket, using a twisting motion. Remove the Sensor; clean and dry the infant's skin. Attach the Sensor pad with new hypo allergenic tape. See Section 3.2, attaching the Respiration Sensor.
3	Monitor has become defective.	Internal fault. Respiration Sensor is defective.	Test by removing the Sensor tube from the Monitor (with a slight turning movement) and by pressing gently with a finger over the inlet socket. This should produce signals on the Monitor. If it does not, the Monitor is faulty and must not be used. Contact nice Neotech Attach a new Respiration Sensor to the Monitor, see Section 2.2, Attaching the Respiration Sensor. If the Respiration Sensor still does not work, the Sensor may be faulty. Contact nice Neotech.
4	No display	Batteries expired. Bad battery connection. Charger defects	Check/recharge the battery. Check battery connections. Replace the charger
5	Keys inoperable/cannot turn unit off	Electronics component failure	Remove the rear panel & disconnect the battery

Note: In the event that none of these procedures produce desired results, remove the batteries and return the unit to nice Neotech for repair